

Application and Training Guide

QuikClot[®] Sport[™] & QuikClot[®] Sport[™] Silver



25 or 50 Gram Bags




What is QuikClot[®] Sport[™] & QuikClot[®] Sport[™] Silver , how do they work, and why use them?

- QuikClot[®] Sport[™] & QuikClot[®] Sport[™] Silver act as a selective molecular sponge. Biologically inert, inorganic, and traps water molecules by hydrogen bond formation in a molecular cage.
- QuikClot[®] Sport[™] & QuikClot[®] Sport[™] Silver cause rapid localized coagulation and the formation of a stable blood clot in a variety of wounds. They do not absorb into the body, and are safe to leave in the wound until further medical care is available.
- QuikClot[®] Sport[™] & QuikClot[®] Sport[™] Silver control bleeding faster than conventional methods.
- QuikClot[®] Sport[™] Silver has Ionic Silver. Silver acts to prevent the growth of bacteria and fungi which can be especially important if medical care is not immediately available.

COOL FORMULA

QuikClot[®] Sport[™] & QuikClot[®] Sport[™] Silver

The formula gives an effective means to control bleeding using Z-Medica's proven technology.
Typical maximum temperature measured: 105⁰F/40.5⁰C

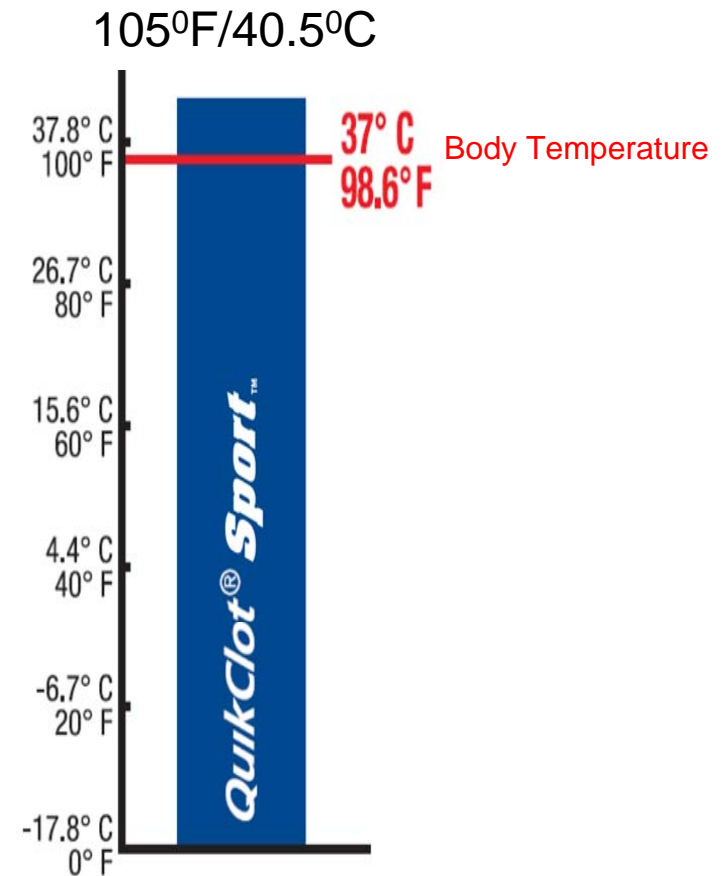

How to use

QuikClot[®] Sport[™] & QuikClot[®] Sport[™] Silver

Read Package Directions before Use.

(Directions are on the back of the product package)

DIRECTIONS FOR USE




1. Open package and remove QuikClot Sport.



2. Place QuikClot Sport over wound and use it to apply pressure directly over bleeding source. (More than one QuikClot Sport may be required.)



3. Continue to apply pressure until bleeding stops.

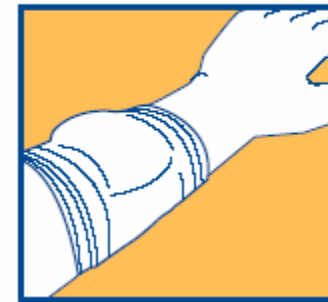

4. Leave QuikClot Sport in place. Wrap to secure in the wound. Do not remove until medical care is available. If wound re-bleeds, apply a new QuikClot Sport. Seek IMMEDIATE medical attention.

Manufactured by:
Z-Medica Corporation
4 Fairfield Boulevard
Wallingford, CT 06492 USA



Emergo Europe

Take Proper precautions when dealing with Blood!

Wear gloves and safety glasses if possible.



Open the QuikClot® Sport™ or QuikClot® Sport™ Silver Package



Place QuikClot® Sport™ or QuikClot® Sport™ Silver over top of the bleeding source.



Quickly apply pressure until bleeding stops.
Suggested time 3 to 5 minutes of continuous pressure



Leave QuikClot[®] Sport[™] or QuikClot[®] Sport[™] Silver in place if necessary. Wrap or bandage to secure the product in the wound. Seek IMMEDIATE medical assistance.

Warning

QuikClot[®] Sport[™] & QuikClot[®] Sport[™] Silver

Are not for internal use