

QUICK START GUIDE



INTUBLADE

**WELCOME
TO THE FUTURE OF
INTUBATION**

INTUBLADE QUICK SETUP

1

ATTACH & PREPARE

- Select Blade: Choose the correct single-use blade.
- Connect Device: Plug IntuBlade into your Android/iOS device via USB-C.
- Launch App: Open the IntuBlade app to view the video feed.
- Do not use after the labeled expiration date.

2

VERIFY

- Check Video Feed: Ensure the live view is clear on your screen.

3

SALINE SPRAY SYSTEM

- Prepare: Insert and twist-lock a 10ml saline-filled syringe into the handle.
- Clear Lens: Press the plunger briefly to clear the lens. Use sparingly.
- Only activate the spray outside the body.

4

RECORD & CAPTURE

- Record: Tap the recording icon to start/stop.
- Capture: Tap the picture icon for snapshots. Files save automatically.

5

DISPOSAL

- Single Use: Dispose of the IntuBlade after each procedure
- Do not reuse or resterilize.

TIPS & TECHNIQUES



WARNING ALWAYS CHECK THE PATIENT'S MOUTH, NOT JUST THE SCREEN, TO AVOID INJURY.

INTUBATION TECHNIQUE

STEP 1: INSERT BLADE

Hold IntuBlade in your left hand; insert along the midline.

STEP 2: VISUALIZE:

Use the video feed to locate the epiglottis; adjust as needed.

STEP 3: GUIDE TUBE

Watch both the mouth and screen while guiding the ETT.

STEP 4: COMPLETE

Finalize tube placement using the video feed for precision.

BLADE INSERTION



QUICK
TIPS

• INSERT GENTLY ALONG THE MIDLINE TO VISUALIZE THE EPIGLOTTIS.

• APPLY ~2 LBS OF UPWARD FORCE TO LIFT THE TONGUE.

• USE A MALLEABLE STYLET TO SHAPE THE ETT FOR EASIER NAVIGATION.



scan to
access
our app &
resources

CONTACT US INTUBLADE

SUPPORT



support@intublade.com

CONTACT



contact@intublade.com

SALES



sales@intublade.com

 Veteran Owned and Operated

WWW.INTUBLADE.COM

OUR MISSION IS CLEAR ©